



WatchCare™ Incontinence Management System

Information that elevates care.

The WatchCare™ Incontinence Management System provides caregivers with real-time incontinence detection and is designed to:




**PROTECT
PATIENT'S SKIN**





**OPTIMIZE CAREGIVER
EFFECTIVENESS**




**IMPROVE PATIENT
EXPERIENCE**

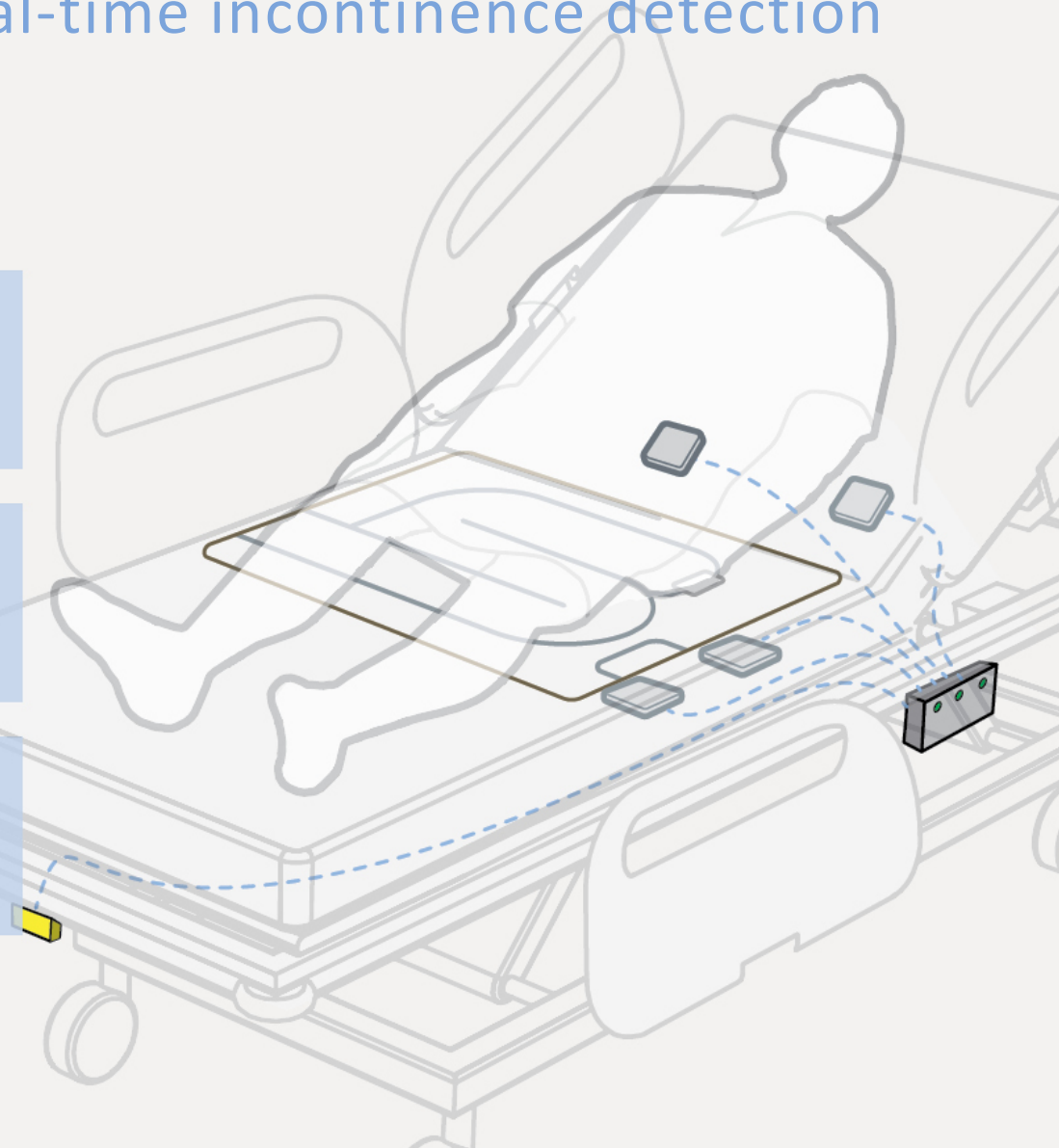

Incontinence is common in the hospital setting.¹

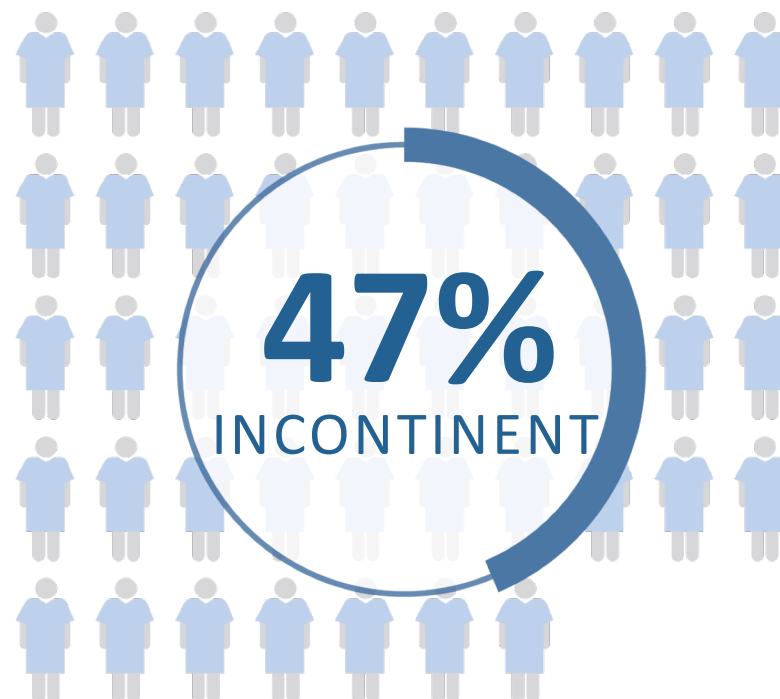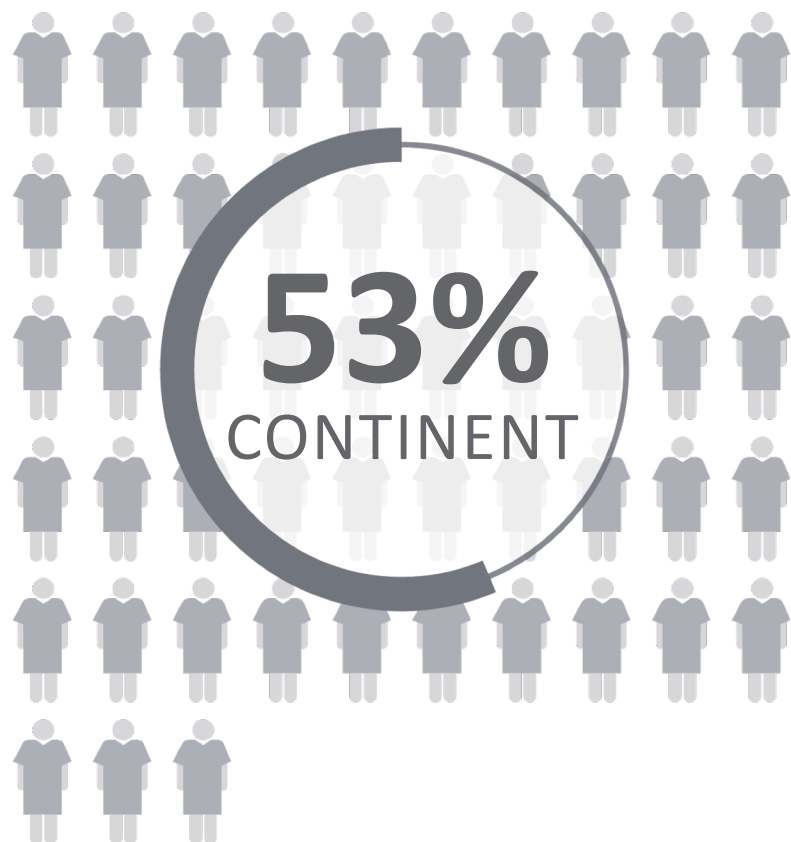


45% of incontinent patients have IAD during their hospital stay:¹

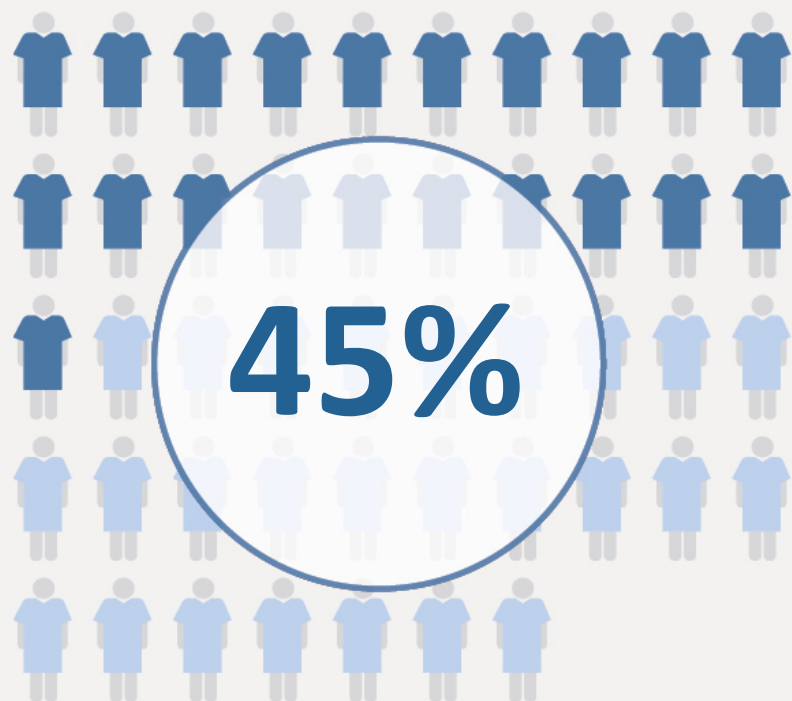


IAD: INCONTINENCE ASSOCIATED
DERMATITIS



29.7% of patients with IAD have urinary incontinence alone:¹

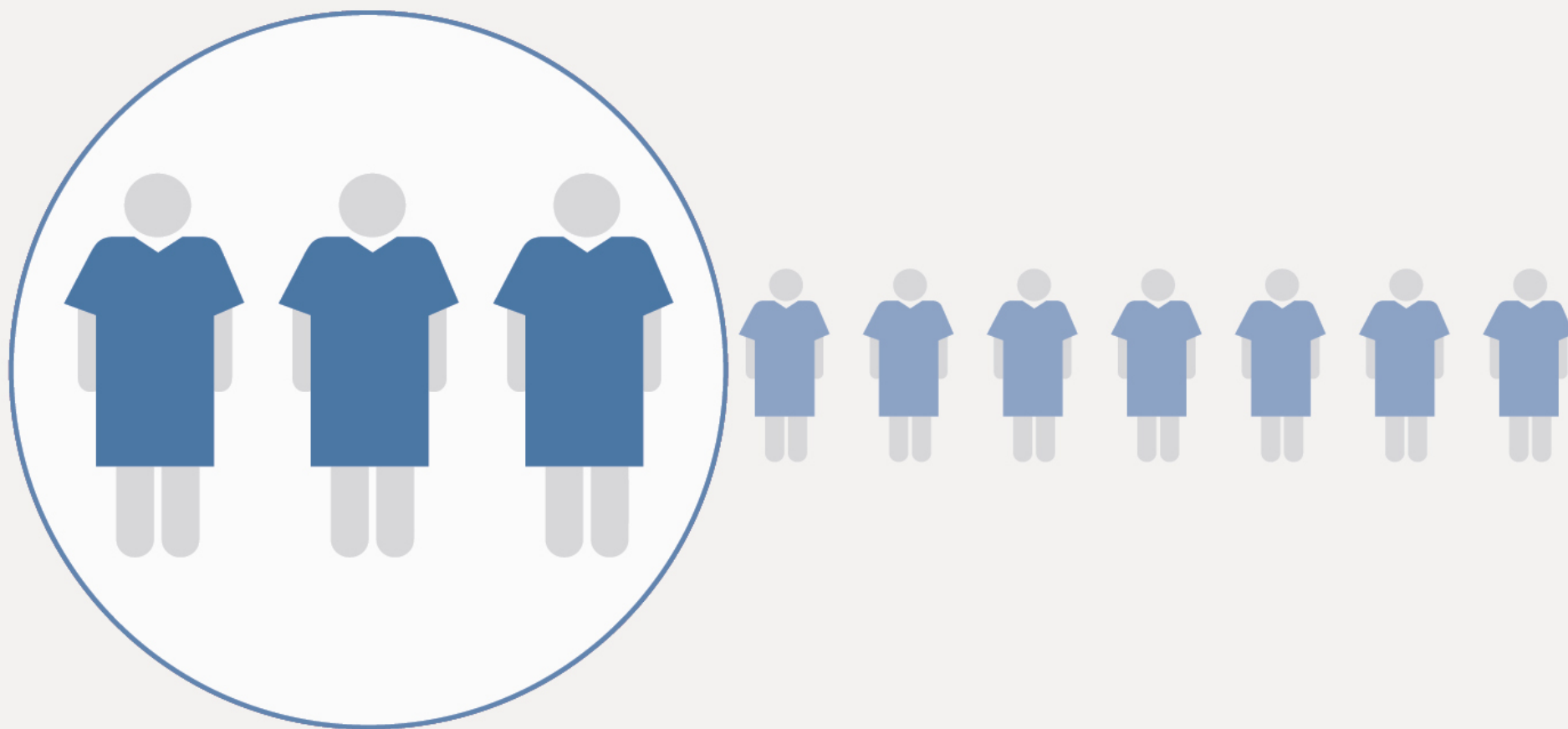

IAD functions as an independent risk factor for pressure injuries:

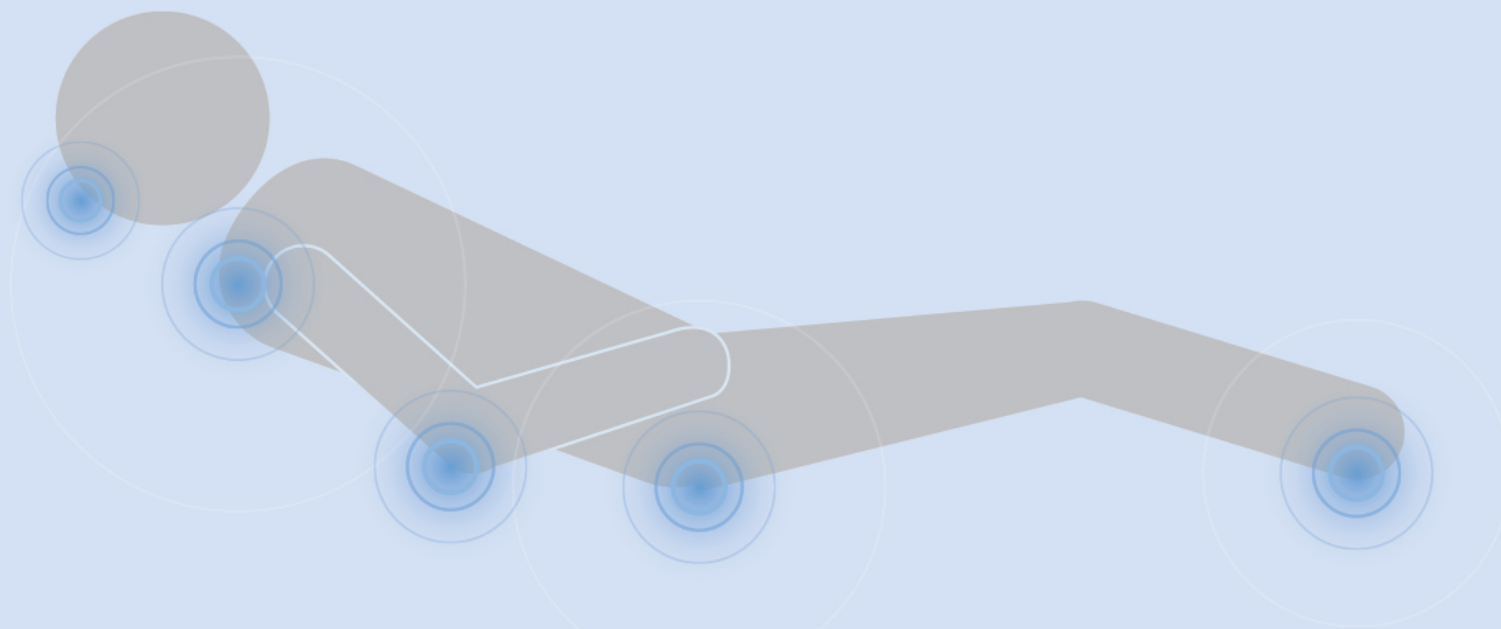

Incontinence Detection and Hourly Rounding

Rounding does **NOT** appropriately address incontinence and incontinence interferes with hourly rounding.



Staff begin to worry about patient experience and skin health 10 minutes after an incontinence event.²

“ WOULD YOU WANT TO LAY IN YOUR OWN EXCREMENT?³

HOURLY ROUNDING IS NOT ENOUGH.⁴ ”

Single Exposure to Clinically Relevant Levels of Moisture Contained by an Incontinence Pad Compromises Skin Health

Studies have shown that *skin is compromised within 15 minutes* of exposure to a single event.⁵

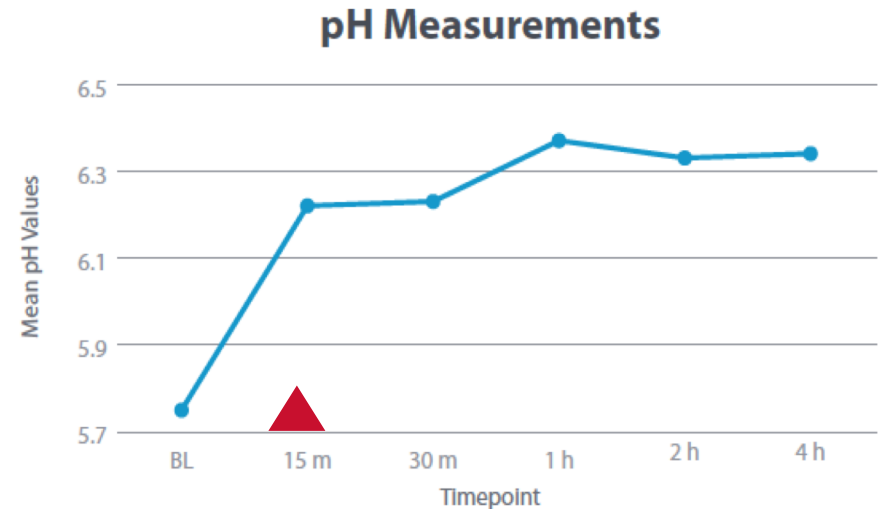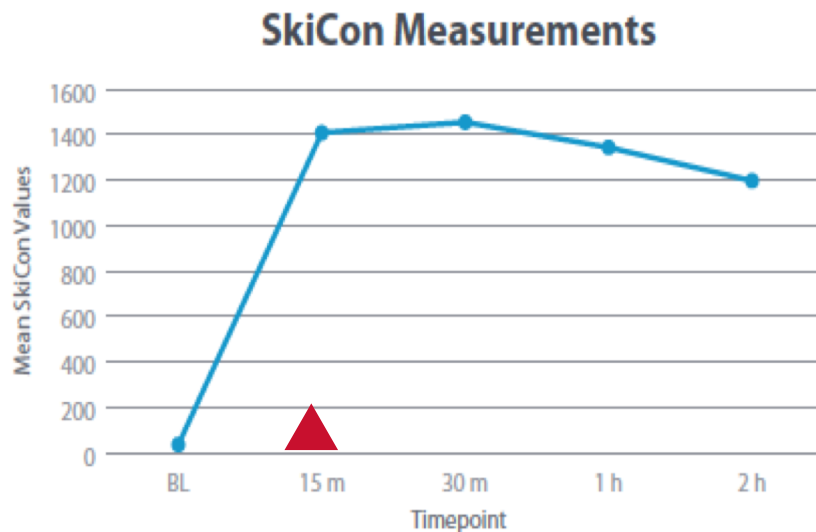

70% experienced skin irritation
WITHIN 1 HOUR.

Evaluation of an Incontinence Management System in the Critical Care Environment: A Case Series

[illegible]

Presented at 2019 NPUAP Annual Conference

Case Study Highlights:


- Neuro-ICU
- Measured 10 patients, 79 incontinence events
- Decreased Exposure Time: 54 minutes → 9 minutes
- No new/worsening cases of IAD
- Provided a less invasive option in comparison to a Foley catheter
- Provided a voice for the patient
- Patients' family members expressed appreciation

Page 8

WatchCare™ Incontinence Management System:


HOW IT WORKS

MONITOR

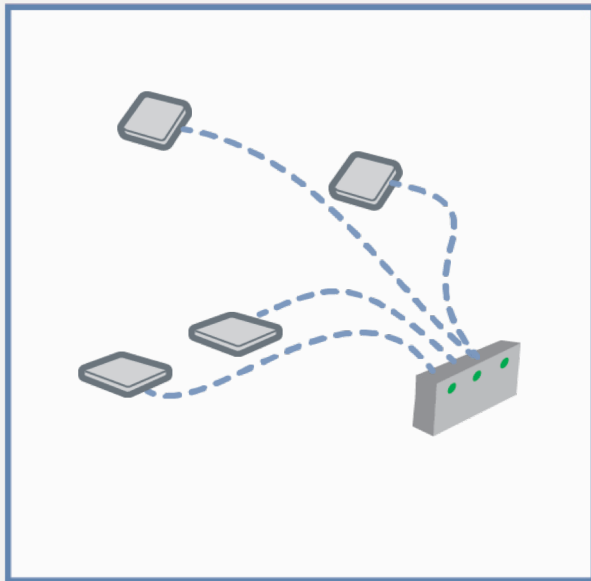

Monitoring system installs under the mattress on VersaCare® and Progressa® beds.

DETECT

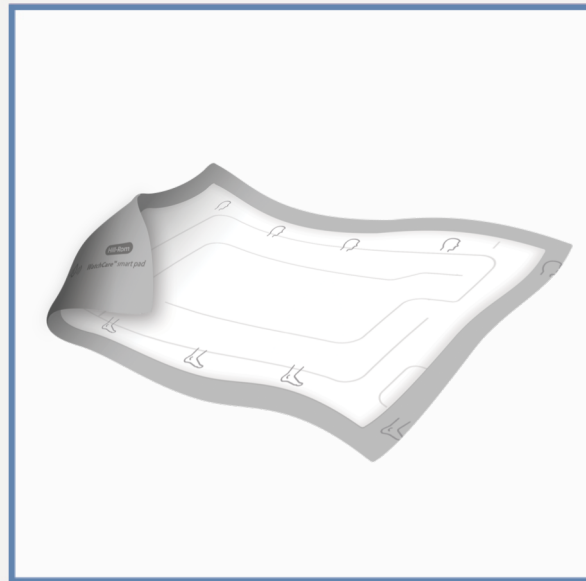

Absorbent smart pads detect moisture and alert the system.


INFORM



Customizable notifications enable prompt, individualized care.

WatchCare™ Incontinence Management System:

HOW IT WORKS

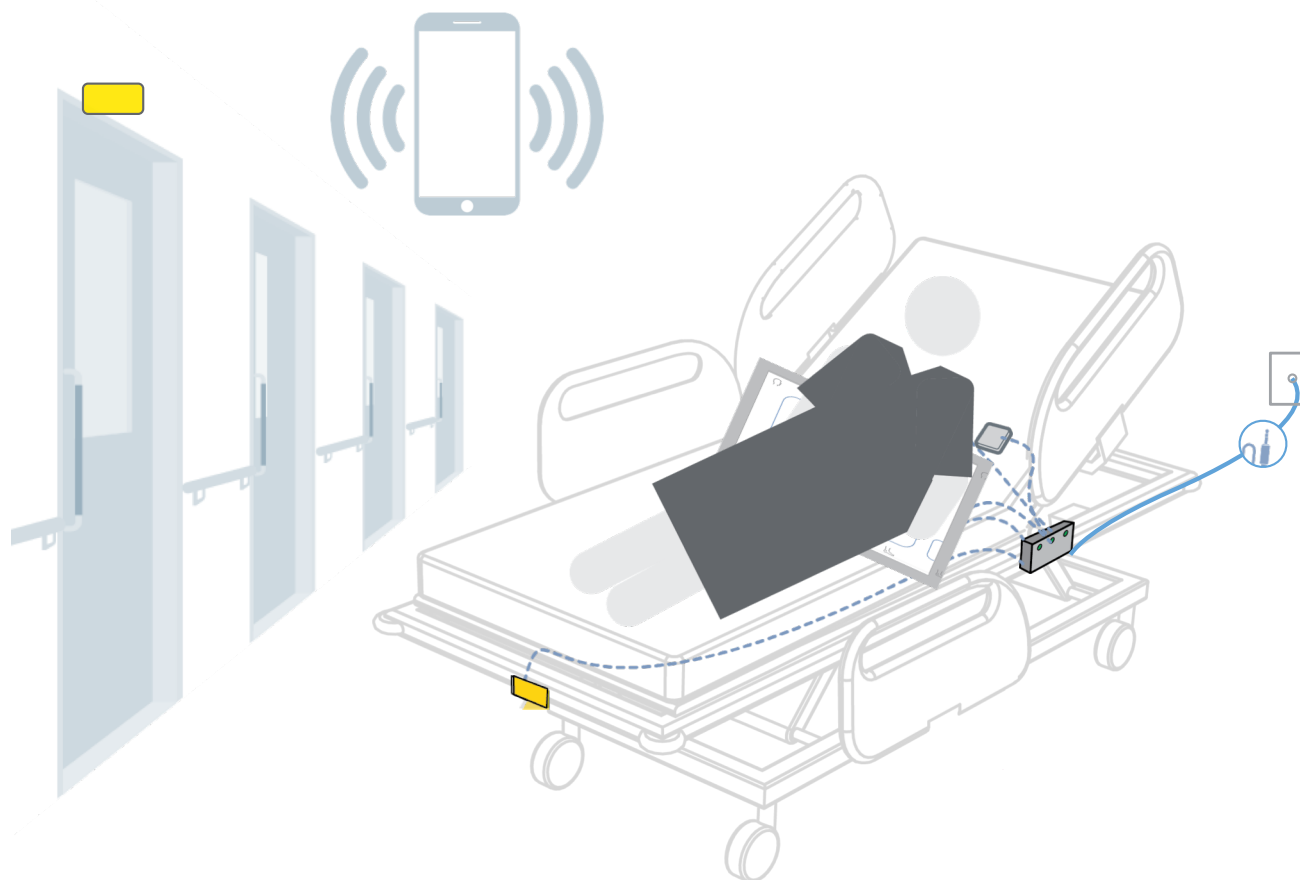

Information that Elevates Care.

References

1. Gray M and Giuliano K. J Wound Ostomy Continence Nurs. 2017;00(0):1-5.
2. Hill-Rom Market Research, Jan 2016, n=202.
3. Product Concept Confirmation, June 2014.
4. Early Concept Feasibility 2015.
5. Phipps L, Gray M, Call E. Single Exposure to Clinically Relevant Levels of Moisture Contained by an Incontinence Pad Compromises Skin Health.