FlexiPort® EcoCuff™

Blood pressure cuff for single patient use

Directions for use

Material Number 722328, 80019369 Ver. A

- Caution** U.S. federal law restricts this device to sale by or on the order of a physician or licensed healthcare practitioner. This device should be used by trained personnel.

Standards

This device is designed to function within the limits prescribed by:

- American National Standard ANSI/AAMI SP10:2002/IR:2008 and ANSI/AAMI SP10:2002/A1:2003, Manual, electronic, or automated sphygmomanometers
- EN 1060-1:2009, Specification for non-invasive sphygmomanometers—Part 1: General requirements.
- EN 1060-2:2009, Specification for non-invasive sphygmomanometers—Part 2: Supplementary requirements for mechanical sphygmomanometers (except for section 7.1.3.1).
- AAMI/ISO 81060-1:2007, Non-invasive sphygmomanometers—Part 1: Requirements and test methods for non-automated measurement type (except for section 7.2.1).

Welch Alllyn Technical Support

www.welchalllyn.com/about/company/locations.htm

	Welch Alllyn, Inc. 4341 State Street Road Skanateles Falls, NY 13153 USA www.welchalllyn.com
	Regulatory Affairs Representative Welch Alllyn Limited Navan Business Park Dublin Road Navan, County Meath Republic of Ireland
	The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

For patent information, please visit www.welchalllyn.com/patents.

© 2013 Welch Alllyn. All rights are reserved. To support the intended use of the product described in this publication, the purchaser of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Alllyn. No other use, reproduction, or distribution of this publication, or any part of it, is permitted without written permission from Welch Alllyn. Welch Alllyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual.

Welch Alllyn®, FlexiPort®, and EcoCuff™ are trademarks of Welch Alllyn, Inc.

ENGLISH

Introduction

Indications for use

The FlexiPort EcoCuff blood pressure cuff is intended to be used on the upper arm in conjunction with noninvasive blood pressure measurement systems. The cuff is nonsterile and is available in pediatric through adult sizes. The device is not intended for neonatal applications. The FlexiPort EcoCuff blood pressure cuff is not designed, sold or intended for use, except as indicated.

Contraindications

Welch Alllyn EcoCuff blood pressure cuffs are contraindicated for neonate use. Do not use on neonate patients.

Warnings

- The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death.

WARNING Intravenous (IV) systems—Do not connect cuffs with luer lock connectors to IV fluid systems. Otherwise, air might enter the patient.

WARNING Do not reuse a cuff on more than one patient. Reusing a cuff on more than one patient might cause cross-contamination.

WARNING Do not clean, disinfect, or sterilize the cuff. These methods have not been tested for effectiveness or impact to cuff performance.

WARNING Use the cuff only when the port falls within the range indicator window on the cuff. Otherwise, erroneous readings might result.

WARNING Do not apply the cuff to areas on the patient where skin is delicate or damaged. Check the cuff site frequently for irritation.

WARNING Do not apply the cuff to an arm used for IV infusion. Inflating the cuff on this arm might interrupt IV flow.

WARNING Minimize cuff movement and arm motion during readings. Excessive movement might affect readings.

WARNING Ensure an airtight seal at all connection points before use. Excessive leaking might affect readings.

WARNING When applying the cuff to the patient, allow space for 1 to 2 fingers between the patient's upper arm and the cuff to ensure proper fit.

Cautions

- The caution statements in this manual identify conditions or practices that could damage the equipment or other property.

Caution Close the hook-and-loop fastener before inflating the cuff. Failure to properly close the fastener might lead to cuff damage.

Caution Keep debris out of the port on the cuff. Debris in the cuff port can damage the cuff.

Caution Intravenous (IV) systems—Do not connect cuffs with luer lock connectors to intravenous fluid systems. Otherwise, fluid might enter the cuff and damage it.

Operation

Notes:

- The cuff works with one-tube and two-tube manual and automated sphygmomanometers.
- The cuff works only with FlexiPort fittings.
- Use the cuff one or more times on one patient and then discard it.
- Do not reuse the cuff on another patient.
- Do not attempt to clean, sterilize, or disinfect the cuff.
- To take a blood pressure measurement:

- To apply the cuff, align the artery marker **➊** on the cuff with the brachial artery and wrap the cuff around the upper arm, so that the part of the cuff with the range indicator window **➋** is on top of the port **➌**.
- Ensure that the port **➌** shows through the range indicator window **➋**, and connect the FlexiPort fitting **➍** to the port through the range indicator window. (If the port does not fall within the range indicator window, select a different cuff size.)
- Take the blood pressure measurement.
- To remove the cuff, disconnect the FlexiPort fitting from the port.
- Unwrap the cuff from the upper arm.

Environmental specifications

- Caution** Store and use the cuff within stated temperature and humidity ranges. Otherwise, the cuff might not meet performance specifications.

Storage temperature	-20°C to 70°C (4°F to 158°F)
Storage relative humidity	15% to 95% (noncondensing)
Operating temperature	10°C to 40°C (50°F to 104°F)
Operating relative humidity	15% to 90% (noncondensing)

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