



Hillrom™

ADVANCING CONNECTED CARE™ WITH ECG CONNECTIVITY

Hillrom and Epic

We're helping you improve clinical workflows and reduce manual documentation for ECG data by sending it right to your EMR. See how the Welch Allyn® Diagnostic Cardiology Suite™ ECG can help create a workflow that works best for your practice.

Unconnected Workflow

- 1 Order an ECG in the EMR
- 2 Manually add patient demographics in the ECG software
- 3 Capture ECG and print test results for physician review
- 4 Add physician notes to ECG printout
- 5 Scan the ECG printout and import into the EMR
- 6 Add diagnosis in the EMR
- 7 Complete order


Connected Workflow

- 1 Order ECG in the EMR
- 2 Launch Diagnostic Cardiology Suite ECG and pass patient demographics from the EMR
- 3 Capture ECG and send a TIFF image to the EMR
- 4 Physician opens Diagnostic Cardiology Suite ECG from the EMR to review, edit and sign the ECG
- 5 PDF report of the final result and discrete data is automatically sent to the EMR

Connectivity Counts

Help save time and improve data accuracy by streamlining your clinical workflows.

- Save time previously spent manually entering data into the EMR
- Improve accuracy by automatically associating results with the EMR orders
- Work more efficiently by freeing up your time for other patient priorities

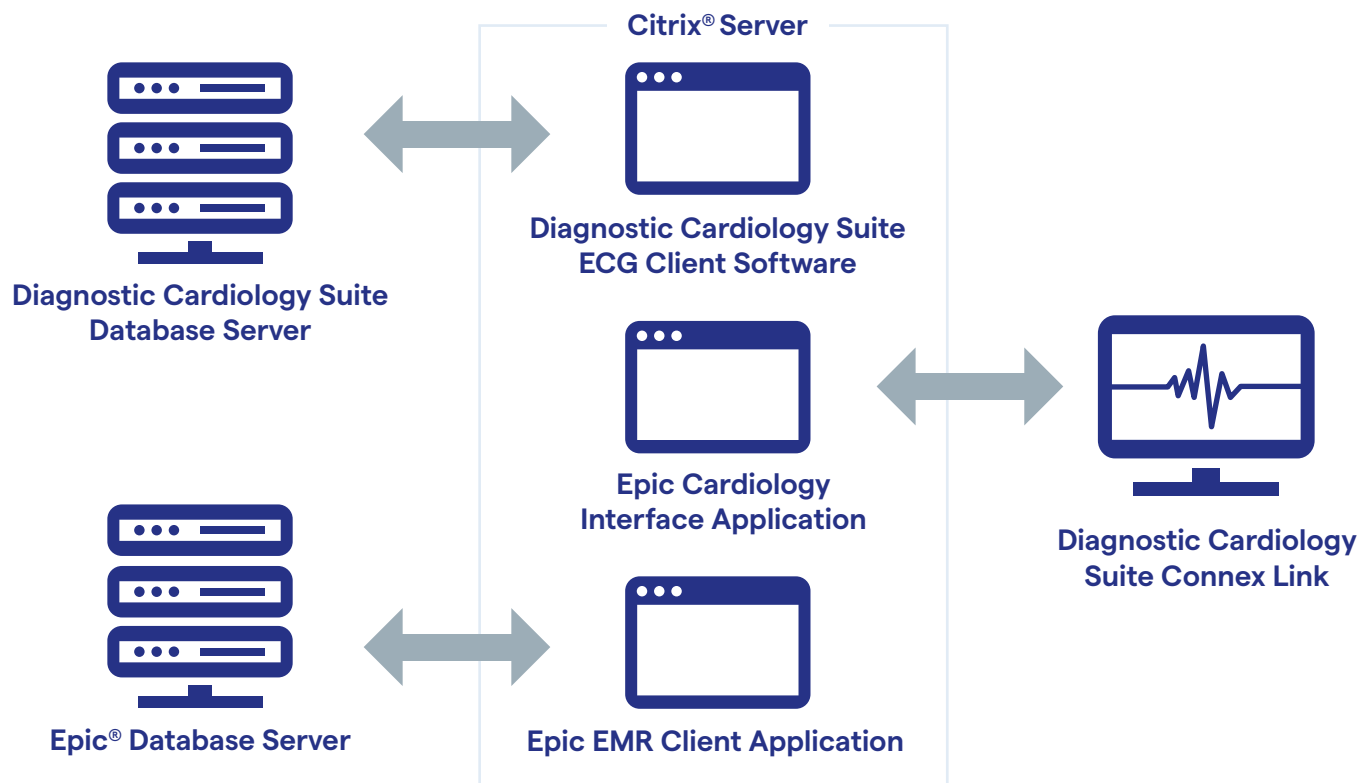

Contact your Hillrom representative to learn more.



hillrom.com

Hillrom partners with IT consultant Polaris Danforth to deliver an HL7® orders/results interface to Epic. Polaris Danforth has core competencies in EMR integrations/interfaces and on-site or remote implementations. Polaris Danforth has certification in Citrix, Epic Ambulatory, Epic Bridges and Epic Identity, making the company a valuable partner in device integration.

Notes on integration:

Hillrom must install and configure the Diagnostic Cardiology Suite ECG before Polaris Danforth starts the interface project.

The Diagnostic Cardiology Suite ECG and Epic Cardiology interface applications are installed on the terminal server running Epic client.

The Diagnostic Cardiology Suite ECG Connex Link must be installed on every thin client that will acquire ECGs.

Hillrom reserves the right to make changes without notice in design, specifications and models. The only warranty Hillrom makes is the express written warranty extended on the sale or rental of its products.