


INSTRUCTIONS


for using Part No.58010
Disposable Sheath for use with
Disposable Vaginal Speculum

1 Pull out sheath and tear off at perforation.

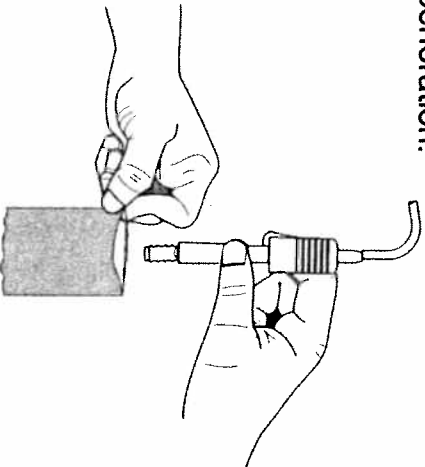


2 Hold open end of sheath and insert light source.


3 Feed bulb end of the light handle thru the opening in the sealed end of the sheath.

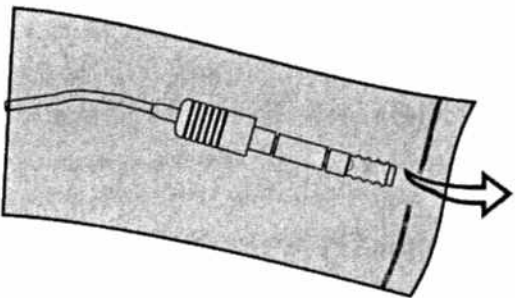

4 Seat the lip on the light handle against the sealed seam on the sheath.

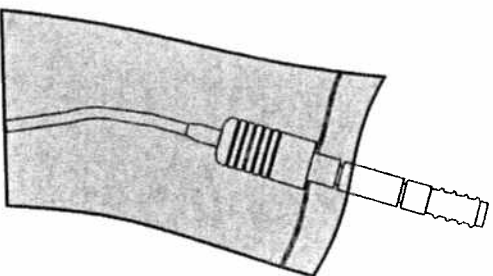

5 Install light handle into speculum.

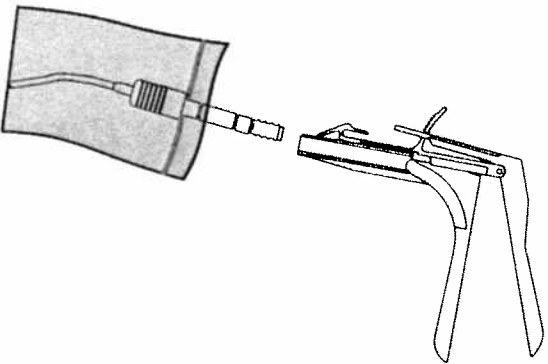

TTND


Welch Allyn

No.58010
DISPOSABLE
SHEATH
FOR

KLEENSPEC[®]
DISPOSABLE VAGINAL
SPECULA
(FOR NO.78000 LIGHT HANDLE)

MANUFACTURED IN THE
USA EXCLUSIVELY FOR
WELCH ALLYN

STATE STREET ROAD
SKANEATELES FALLS, N.Y. 13153-0220

NOTE: This product does not contain latex rubber.
580109A 8/94